

Instructions for Use



Placement of Foot Drop Strap Across Metatarsal Heads



Placement of Anti-Rotation Wedge



Opening Tabs for Heel Assessment



Used of Extension Straps as Needed





ROOKE® HFS™ BOOT

Heel Float System

Trimming the Boot to Fit Smaller Feet

Cut Outside Dark Line



ROOKE® HFS™ BOOT

Heel Float System

Access for SCD Tubes--**Cut Between Lines**



Optional: Cut **Between the Lines** to Open for Temperature Control or Wound Assessment



Osborn Medical Corporation www.RookeBoot.com

800.535.5865 – Fax 507.932.5044 – 100 West Main Street Utica, MN 55979